Installation and Operation

NeoScan™ 2000

Dear Valued Customer,

We are delighted that you have made a purchase of the NeoScan™ 2000 and we assure you that your satisfaction is our top priority.

To help you get started, we have included a quick guide along with this manual. This handy reference will provide you with essential information and instructions to make the most of your product.

For more detailed information and comprehensive guidelines, we encourage you to download the Instruction for Use (IFU) from our website. Simply visit www.neoss.com/neoscan/download to access the IFU.

Should you encounter any issues or have any questions regarding your purchase, our dedicated support team is ready to assist you. You can contact our local customer support, the details of which are outlined below. Alternatively, feel free to reach out to your local distributor for assistance.

We hope that your purchase brings you great satisfaction and fulfills your expectations. Should you require any further assistance, please do not hesitate to reach out to us.

Best regards,
Neoss Group

Customer support

For support, please visit our webpage:
neoss.com/neoscan#support

Email support

DE: neoscan.support.de@neoss.com

DK: neoscan.support.dk@neoss.com

ES: neoscan.support.es@neoss.com

FR: neoscan.support.fr@neoss.com

IT: neoscan.support.it@neoss.com

NL: neoscan.support.nl@neoss.com

NO: neoscan.support.no@neoss.com

SE: neoscan.support.se@neoss.com

UK: neoscan.support.uk@neoss.com

US: neoscan.support.us@neoss.com

Other: neoscan.support@neoss.com

NeoScan™ 2000

Notice

The information in this document is subject to change. Neither Neoss AB nor Neoss Ltd or any of its subsidiaries shall be liable for errors contained herein or for incidental damages in conjunction with the furnishing, performance, or use of this material. No part of this publication may be reproduced without the permission of the manufacturer.

All trademarks and registered trademarks are the property of their respective holders.

The NeoScan™ 2000 is intended for professional use only.

U.S. Federal law restricts this device to sale by or on the order of a dentist.

Manual Name: NeoScan 2000 User Manual:
Installation and Operation
Part Number: 16004

Revision: 1

Date: 2023-09

1 – Safety information		5
	Conventions	5
2 – NeoScan 2000 overview		6
	Nomenclature	6
	Computer system requirements	7
	Charging the battery	7
	Charging holder overview	8
	Wireless connection	9
3 – Setting up the NeoScan 2000		10
	Setting up the scanner	10
	Using the scanner charging holder	10
	Installing the desktop charging holder	11
	Installing the wall mount charging holder	12
4 – Getting started		13
	Accessing the software user interface	13
	Audio prompts	14
	Preparing the teeth	14
	Preparing the scanner	14
	Starting scanning	15
	Scanning approach	15
	Scanning protocol	15
5 – Maintenance		16
	Cleaning, disinfection, sterilizing	16
	Clean and disinfect the scanner	16
	Clean and sterilize the scanner tip	17
	Precautions before use	18
6 – Troubleshooting		19
	NeoScan 2000 troubleshooting instructions	19

Conventions

The following special messages emphasize information or indicate potential risks to personnel or equipment.

WARNING: Avoids injury to yourself or others by following the safety instructions precisely

CAUTION: Alerts you to a condition that might cause serious damage or cause problems.

NOTE: Provides extra information and hints.

WARNING: We recommend that you consult this document before using the device.

All trademarks and registered trademarks are the property of their respective holders.

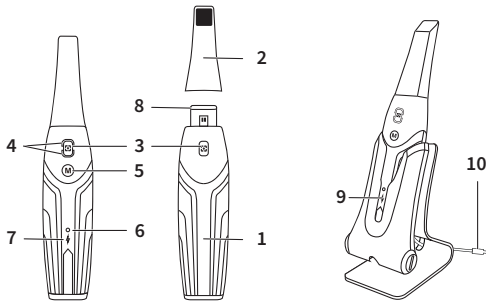
The device is intended for professional use only.

U.S. Federal law restricts this device to sale by or on the order of a dentist.

If any serious incident occurs in relation to the device, the user must report it to Neoss AB and to the competent authority of its Member State in the European Union.

2 – NeoScan 2000 overview

Nomenclature

1 – Handpiece

2 – Scanner Tip

3 – Scan Button

Press once to start scanning, press again to stop scanning

4 – Mode Indicator

Upper jaw scan mode

Lower jaw scan mode

Buccal bite registration mode

5 – Mode Switch Button

Press this button to switch between different modes

6 – Status Indicator

Blinking slowly: starting up

Blinking rapidly: preparing network

Breathing: ready to be connected to the software

On: Connected to the software and selected as active scanner

7 – Battery Indicator

Solid green: battery level higher than 50%

Solid yellow: battery level between 20% and 50%

Blinking yellow: battery level lower than 20%

Breathing green: charging and battery level higher than 50%

Breathing yellow: charging and battery level lower than 50%

Solid green: fully charged

8 – Lens Window

9 – Charging Holder

10 – USB Cable (1 m length)

Computer system requirements

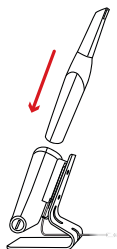
For the computer system requirements, see the **NeoScan™ 2000 User Manual: Safety, Regulatory and Technical Specifications**.


It is **MANDATORY** to check that your computer system configuration is compatible with the computer system requirements for the NeoScan 1000 software.

Charging the battery

A fully charged battery can support up to 2 hours of scan time. When the Battery Indicator flashes yellow, it means the battery is low and needs to be charged in time. To charge, the scanner needs to be inserted into the Charging Holder.

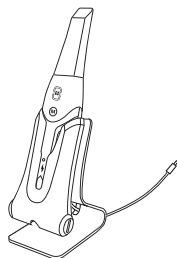
The Charging Holder must be connected to a USB Type C power source to be functional.


You can also continue to use the scanner by replacing the battery, follow these steps to replace the battery.

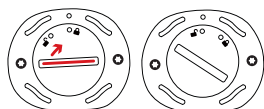- Turn the battery cover with a coin to unlock.

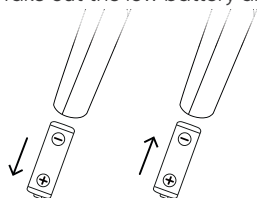
- Take out the low battery and insert a charged one.


WARNING: Please use the battery specified (Li-18650-3.6V 3500mAh-PCM-Cap).

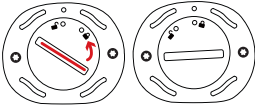- Close the battery cover by turning it to lock.

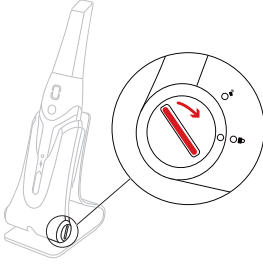- The Charging Holder holds a spare Battery and keeps the battery level at approximately 80%. Open the battery cover by turning it to unlock and you will be able to take the battery out.


NOTE: Please fully charge the battery before use.

Charging holder overview


Place the scanner in the Charging Holder when you are not using it.

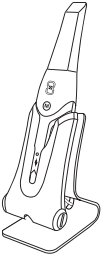NOTE: The scanner will automatically shut down if it is idle for more than 3 minutes. If you need to use it again, press any button on the scanner, or take it out of the Charging Holder.


Wireless connection

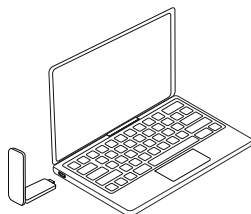The scanner connects to the software through a wireless connection. Before using the scanner, a compatible wireless adapter needs to be connected to the computer that runs the software.

NOTE: Expand the wireless adapter to maximize the network signal strength.

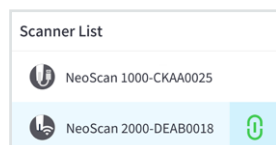After the compatible wireless adapter is connected, Select the scanner you want to connect from the Scanner list window at the bottom right of the software and click the connect button.

Indicates a wireless scanner

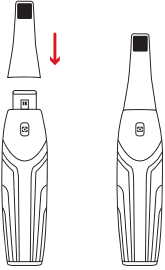
Indicates a wired scanner

3 – Setting up the NeoScan 2000

Setting up the scanner

To set up the scanner, follow these steps:

- Visit www.neoss.com/neoscan/download and download the installation file and instructions according to the product model.
- Double-click the software installation file.
- Choose a language from the drop-down list and click Ok to install.
- Follow the instructions on the screen to complete the installation.
- Firmly slide the Scanner Tip onto the end of the scanner.

- Short press any button to start the scanner.
- After the scanner is powered on, open the software and connect the scanner through the wireless network.

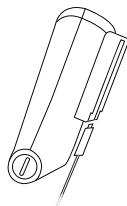
Using the scanner charging holder

The Charging Holder can be used standing on a flat plane or mounted on the wall.


Installing the desktop charging holder

To place the Charging Holder on a desktop, follow these steps:

- Select a clean surface area.
- Insert one end of the charging cable into the bottom of the charger.

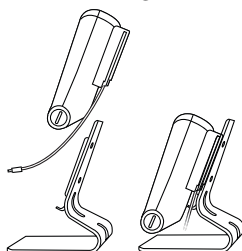
- Insert the charger with the connected charging cable into the base.


- Insert the charging cable into the holder bracket.

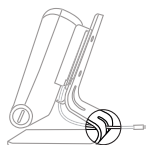- Pass the other end of the cable through the hole reserved on the base.

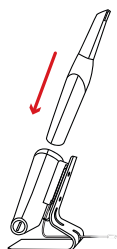
- Insert the scanner into the Charging Holder.

Installing the wall mount charging holder

To mount the Charging Holder on the wall, follow these steps:

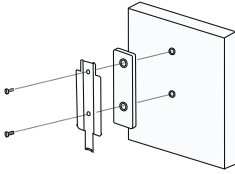
- Select an area that you can access easily.
- Insert the screws through the holes in the holder bracket to affix it to a solid surface.


Insert one end of the charging cable into the bottom of the charger.

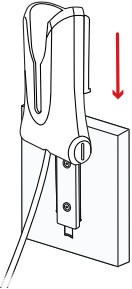Insert the Charging Holder into the holder bracket.

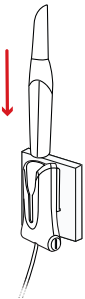Insert the scanner into the Charging Holder.


CAUTION: If the Charging Holder is not properly installed, there is a risk that the Charging Holder can fall off the wall, resulting in damage to the scanner.


Accessing the software user interface

To access the software user interface, follow these steps:

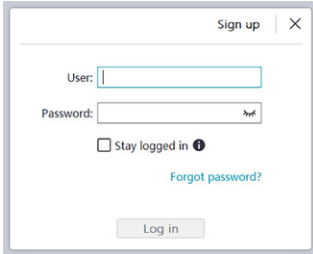- Launch the software, the software will automatically pop up the login window.


- If you don't have an account, click the **Sign up** button to register the organization, and complete the email verification.
- Type your account information in the username and password fields, click the **Log in** button.
- The Patient Information dialog box will pop up.
- Do one of the following:
 - Enter patient information and tooth number information and click **OK**.
 - Click **Skip** and continue without patient information.

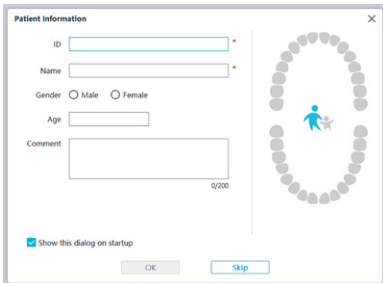- Click **Connect** button to connect wireless scanner in Scanner List.

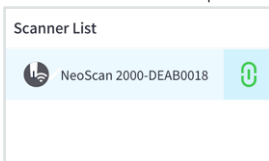If the scanner is not activated, the scanner activation dialog will be displayed. Follow the instructions on the screen to complete the scanner activation.

- Click the Option menu button  and select Preferences.
- Customize the configuration options.
- You can now start acquiring 3D models.

Audio prompts

Scanning sound

When the **Enable scanning sound** option is enabled, your computer will play a continuous sound when you are successfully scanning. If the sound stops, it means the scan has stopped. If you need to continue, please return to the previous scanning area until the scanner resumes scanning and your computer plays a continuous sound. When a bite registration relationship is successfully scanned, your computer will also play a short sound.

Warning sound

When the **Enable warning sound** option is enabled, if the cumulative scanning time of the current case exceeds the recommended threshold and your computer may not be able to maintain the peak scanning performance, your computer will play a short warning sounds.

NOTE: If your computer does not have speakers, these options will not take effect.

Preparing the teeth

- If there is a preparation area, retract the gingiva by gingival restriction cords. And extract the cords just before scanning the preparation.
- Before starting the scan, dry the teeth thoroughly.
- During the scan, re-dry the teeth moderately.

Preparing the scanner

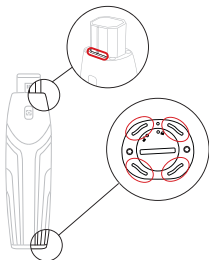
The Scanner Tip attached to the Handpiece provides sanitary shield for patients. Always disinfect the body of the scanner and perform sterilization on the Scanner Tip after each use.

CAUTION: Scanner Tips are shipped non-sterile. You must sterilize them before the first use.

CAUTION: Avoid any liquid from leaking into the air outlet near the Scanner Tip mount or the air inlet at the rear of the Handpiece, otherwise the Handpiece may be damaged.

To prepare the scanner, follow these steps:

- Make sure the Lens Window at the base of the scanner is clean by wiping it with a moist, lint-free cloth or lens tissue.
- Slide the Scanner Tip onto the scanner as shown below.

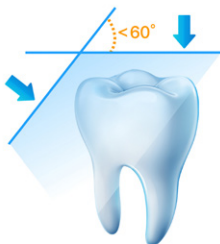
Starting scanning

To start scanning, place the Scanner Tip on the surface of the tooth to stabilize the scanner and press the Scan button. Wait until a 3D model appears on the 3D model display screen, and then slowly move it along the arch at 0–5mm from the teeth.

Scanning approach

The recommended scanning method is to start with a molar, since it has greater details for easier identification. Change the scanning angle to less than 60 degrees during scanning to allow the surfaces to overlap, if the overlap is too small, the alignment may be lost.


Scanning protocol

The recommended scanning protocol consisting of 3 sweeps: occlusal, lingual and buccal to ensure good data coverage of all surfaces.

It is recommended to start the first sweep from the occlusal surface, and you should start with the first molar.

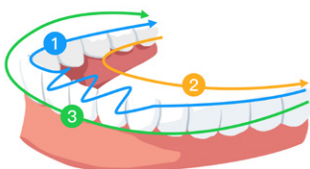The second sweep can scan both the lingual and buccal sides, and the third scan covers the opposite side of the second sweep.

5 – Maintenance

Cleaning, disinfecting, sterilizing

Perform the following maintenance activities on your scanner and accessories regularly.

To ensure maximum hygienic safety for the patient, carefully follow the instructions to prepare the scanner for use.

To ensure maximum hygienic safety for the patient and to minimize the risk of cross-contamination, carefully perform the following maintenance activities on your scanner and accessories.

After each patient:

- Clean and disinfect the scanner. See “Clean and Disinfect the Scanner”.
- Clean the Scanner Tip, and then perform Autoclave Sterilization (See “Clean the Scanner Tip” and “Sterilize the Scanner Tip”).

There are 2 models of Scanner Tip:

Model	Size	UDI-DI	Manual Cleaning	134° C Sterilization
TP102N	Large	(01)06973993441034	✓	✓
TP103N	Small	(01)06973993441041	✓	✓

Clean and disinfect the scanner

General Warnings

WARNING

- Read and follow the warnings and personal protection instructions provided in the Safety Data Sheet (SDS) for the disinfectant used to process the scanner.
- You must wear gloves while cleaning and disinfecting the scanner.
- The scanner must be disinfected with a recommended intermediate-level disinfectant solution with tuberculocidal activity between patients.
- **DO NOT** use a disinfectant containing phenolics or iodophors; doing so will damage the surface coating of the scanner.
- Never put the scanner in an autoclave device or immerse it in water or the disinfectant solution.
- Excessive fluids can damage the scanner.
- Do not use cotton, cloth, or tissues soaked with disinfectant to disinfect the scanner.

Clean the Scanner

- If the scanner is visibly contaminated with blood and/or body fluids, you must clean it before disinfecting it.
- To clean the scanner, follow these steps:
 - Dampen (do not soak) a lint-free cloth with lukewarm water.
 - Remove the blood and/or body fluids with the dampened lint-free cloth.

Disinfect the Scanner

After each patient, the scanner must be thoroughly disinfected.

To adequately disinfect the scanner, follow the disinfectant manufacturer's instructions for the appropriate contact time.

CAUTION: If the scanner is visibly soiled, it must be thoroughly cleaned prior to disinfecting. See "Clean the Scanner"

To disinfect the scanner, follow these steps:

- Remove the Scanner Tip.
- Remove all visible soil (see "Clean the Scanner").
- Use a commercially prepared intermediate level disinfectant wipe. Follow the manufacturer's instructions for contact time.

Recommended disinfectant wipes: CaviWipes™*

WARNING: Using a disinfectant that has not been approved may cause damage to the scanner.

- Thoroughly wipe all surfaces of the scanner, **DO NOT** allow liquid to enter through the gap, air outlet, or pin holes.

WARNING: Do not rinse.

- Allow to air dry.
- After the scanner has dried, use a clean, lint-free cloth dampened with water to remove residual disinfectant from the surface of the scanner.

Clean and sterilize the scanner tip

WARNING

- Wear gloves when handling a contaminated Scanner Tip.
- Read and follow the warnings and personal protection instructions provided in the manufacturer's SDS for the detergent used to clean the Scanner Tip.
- Do not soak the Scanner Tip in disinfectant for a long period.
- Dry the Scanner Tip thoroughly before mounting onto the scanner.
- Do not use an ultrasonic cleaning machine to clean the Scanner Tip.
- Do not soak the Scanner Tip with alcohol-based disinfectants.

Clean the Scanner Tip

To manually cleaning the Scanner Tip, follow these steps:

- Rinse excess soil from the Scanner Tip (2 minutes).
- Using a soft brush, apply enzymatic detergent solution (e.g., Metrex EmPower) to all surfaces.
- Rinse under clean, running water (2 minutes).
- Inspect Scanner Tip. If the Scanner Tip is not clean, repeat the last three steps.
- Use a lens tissue or lint-free cloth to remove any dust from the mirror in the Scanner Tip.

* CaviWipes are a trademark of Metrex Research, LLC.

Sterilize the Scanner Tip

Scanner Tips shipped non-sterile. You must sterilize them before use.

CAUTION: If you limit the exposure time at 134°C to not more than 6 minutes, you can autoclave the Scanner Tip up to 60 cycles.

To sterilize the Scanner Tip, follow these steps:

- Put the Scanner Tip into a sealed steam sterilization pouch.
- Place the Scanner Tip in a steam autoclave for sterilization.
 - For TP102N and TP103N, exposure temperature should be set as 134°C.
 - For TP102N and TP103N, exposure time should exceed 3 minutes.
 - For TP102N and TP103N, exposure time should not exceed 6 minutes.

Precautions before use

Perform the following activities on your product and accessories before use.

Visually Inspecting the Scanner for Damage

Visually inspect the scanner for damage or signs of deterioration by doing the following:

- Inspect the Lens Window.
 - Inspect around the buttons and contact points.

If damage is noted, do not use the scanner and contact your representative.

Visually Inspecting the Scanner Tip

Visually inspect the Scanner Tip for signs of deterioration by doing the following:

- Verify that the Scanner Tip is not damaged, and its components are not detached.
- Verify that the Scanner Tip mirror does not have any smudges or scratches on it.

If deterioration is noted, replace the Scanner Tip.

WARNING

- The Lens Window on the scanner is a delicate optical component. Mount the front Protective Cover to protect the Lens Window from damage and dirt when the scanner is not in use.
- The mirror in the Scanner Tip is a delicate optical component. Its clean and undamaged surface is critical to scan quality.

In the event that you see poor scan quality or an unclear video preview in the software, clean the Scanner Tip mirror and the Lens Window of the Handpiece using a microfiber cleaning swab, applying ethanol that is free of impurities.

NeoScan 2000 troubleshooting instructions

Problem Description	Action
There is mismatching and overlap on the 3D model.	Remove mismatched data and excessive tissue using the Cut tool and rescan.
After bite registration, there is a gap or intersection between the upper jaw and the lower jaw.	<ul style="list-style-type: none"> • Set "Bite adjustment" to "Avoid intersections" in Preferences. • Delete the incorrect bite view, and rescan.
Precision degradation is observed, or images are not well-stitched during acquisition.	Ensure that the Lens Window at the base of the scanner is clean by wiping it with a moist, lint-free cloth or lens tissue. Use a lens tissue or lint-free cloth to remove any dust or water stains. Make sure the Scanner Tip is firmly installed and there are no dark edges on the live video.
Reconstruction of metallic preparations is sometimes difficult.	<ul style="list-style-type: none"> • Adjust the scanner position (for example: distance or angle) and scan more of the area. • Move the surgical light away from the patient to decrease light scatter. • Turn on "Shining Surface" button.
The Scanner Tip is installed but not detected. No live video is displayed, and the Scanner Tip not detected icon is displayed at the bottom-right of the interface.	Reinstall the Scanner Tip, and make sure the Scanner Tip is in firm contact with the scanner.
Fogging appears on the inner surface of the lens window at the base of the scanner.	<ul style="list-style-type: none"> • Mount a completely dry tip on the scanner, and place the scanner in the holder or set it on the desk, and wait until the fogging fades. If the fogging does not disappear completely after 24 hours, contact your local service provider for assistance. • Ensure that the tip is thoroughly dry before mounting on the scanner, and do not use a cloth soaked in disinfectant to clean the scanner.
The scanner does not cast light, and a static preview image is displayed in the video preview screen.	Close the software, and then launch the software again.
Scanner is not shown in the 'Scanner List' in the software while the Status Indicator keeps blinking in breathing mode.	Reset the network in the scanner. To do so, while the Status Indicator is in breathing mode, simultaneously press the Scan Button on both sides for two seconds and release. The Status Indicator should start blinking rapidly, then enter breathing mode. The device should then be shown in the 'Scanner List' in the software.

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
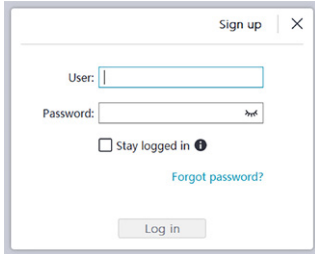
The image shows a 'Sign up' dialog box with the following elements: a title bar with 'Sign up' and a close button; a 'User:' text input field; a 'Password:' text input field with a 'type' dropdown; a 'Stay logged in' checkbox with an information icon; a 'Forgot password?' link; and a 'Log in' button at the bottom.

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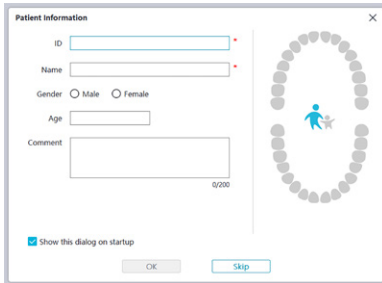
The image shows a 'Patient information' dialog box with the following elements: a title bar with 'Patient information' and a close button; 'ID' and 'Name' text input fields; 'Gender' radio buttons for 'Male' and 'Female'; an 'Age' text input field; a 'Comment' text area with a '0/200' character count; a 'Show this dialog on startup' checkbox; and 'OK' and 'Skip' buttons at the bottom. On the right side, there is a circular diagram of teeth with a blue person icon in the center.

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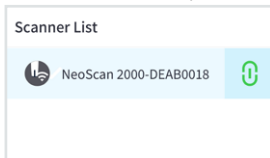The image shows a 'Scanner List' dialog box with a single entry: 'NeoScan 2000-DEAB0018'. The entry has a status icon on the left and a green checkmark icon on the right.

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Distributed by:

Neoss Ltd. and its subsidiaries as above

Manufacturer:

Alliedstar Medical Equipment Co., Ltd. No.1-4, Floor 3, Unit 2, Building 3, No. 222, West third section, Waihuan Road, Yanjiang District, Ziyang, 641300, Sichuan, P.R. China | Email: support@allied-star.com

EU Authorized Representatives:

SUNGO Europe BV., Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands

EC REP

Neoss AG

The Circle 23, 8058 Zurich-Airport, Switzerland

E: distributors@neoss.com

E: info@neoss.com

For support, please visit our webpage:

neoss.com/neoscan

NeoScan customer support: +44 1423 533 991

Contact your local representative for product availability details. All products are not available in every market.

16004_1 EN, DA, FR, DE, IT, NO, PT, ES, SV 2023-09 © Neoss Group, 2023. Copyrights, design rights and trademarks, Neoss documents, software and designs may not be reprinted, copied or published in whole or part, without the written authorization of Neoss Group. Neoss and the N logo are registered trademarks of Neoss Group. NeoScan™, NeoPro™ and NeoConnect™ are trademarks of Neoss Group. CaviWipes are a trademark of Metrex Research, LLC.

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Neoss AB, Arvid Wallgrens backe 20, SE-413 46 Göteborg | Phone: +46 (0)31 88 12 80 | Email: info@neoss.se